

Secure Positioning

The insert is designed to remain securely in place and provides a complete seal, thanks to the bulb and external retainer.

24 Hour Use

It may be worn continuously for up to 24 hours, allowing you to worry less and do more of what you love.

Ease-of Use

The applicator helps ease insertion.

Flexibility and Comfort

Made from soft silicone and filled with liquid, the insert is carefully crafted to ensure comfortable insertion and to adapt and conform to the movement of your body.


Hygienic

The insert features a protected insertion sleeve, intended to increase hygiene.


A SOFT INSERT DESIGNED FOR COMFORT

The Navina Insert is a soft and flexible insert that is engineered to provide a complete seal for up to 24h in order to prevent bowel leakage. The insert is intended to be hygienic, and with its body-conforming silicone shape, it's designed to offer comfort, so that you can feel secure regardless of what you are doing.

How to use

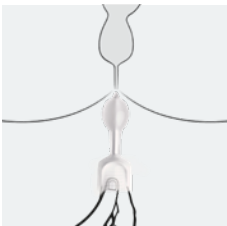


1. Hold the pre-lubricated insert by the white plastic applicator and align the tip of the bulb with the anus.

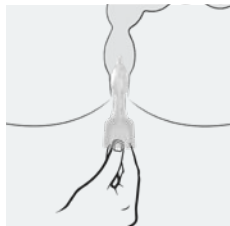

2. Gently push the insert until the broader retainer rests on the anus. Withdraw the applicator and discard it in the regular trash.



3. The liquid-filled insert adapts to the contours of the body and helps prevent leakage.

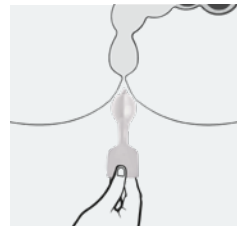

4. To remove it, grasp the retainer and gently pull to remove the insert and discard it with the regular trash.

Reference	Product	Specification	Uses
6930400	Navina Insert, Standard	30 pieces	Single use
6930500	Navina Insert, Large	30 pieces	Single use
6930600	Navina Insert, Starter Pack	5 Standard and 5 Large pieces	Single use



Navina Bowel Care

We know bowel care is not a one size fits all, which is why we take pride in being able to offer a range of solutions to meet each individual's needs, all under one brand that promises you the same quality, ease of use and flexibility regardless of product choice.

At Wellspect we develop innovative continence care solutions that improve quality of life for people with bladder and bowel problems. We inspire our users to build self-confidence and independence as well as good health and well-being. We have been leading the industry for over 40 years with our product brands LoFric® and Navina™. We always aim to minimize the environmental impact of our products and passionately strive to become climate neutral. We work together with users and healthcare professionals to improve clinical outcome in a sustainable way, now and for the future.

Wellspect. A Real Difference.

For more information about our products and services, please visit [Wellspect.com](https://www.wellspect.com).

Join the conversation on LinkedIn, Twitter, Facebook and Instagram.

[wellspect.com](https://www.wellspect.com)

Wellspect HealthCare, Aminogatan 1, P.O. Box 14, SE-431 21 Mölndal, Sweden. Phone: +46 31 376 40 00.

Navina Fecal Incontinence Insert



Rx Only



Manufacturer:
Minnesota Medical Technologies
2446 Henry Road Northwest
Stewartville, MN 55976 USA
507-533-0366
www.mnmedicaltechnologies.com
540035 Rev 1

Importer:
Wellspect HealthCare
Aminogatan 1
P.O. Box 14, SE-431 21
Mölndal, Sweden.
www.wellspect.com

MPS Medical Product Service GmbH
Borngasse 20
Braunfels 35619
Germany